

ULNA

Electric massage bed Ref. 2238TB



High-performance multifunctional bed, especially suitable for beauty centers. Equipped with 3 motors that control height, backrest and central module inclination. The base is made of a metallic structure in a wenge colour. The upholstery is made from white polyurethane of a high quality that eases cleaning.

WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the bed. If an inspection becomes necessary, please contact your supplier or an authorized technician.
- Never use the bed in wet or poorly ventilated environments, such as bathrooms to prevent from damaging the electrical components of your bed.
- Never use the bed outdoors; the exposure of the elements could damage the bed or reduce its lifespan.
- If the power cord was damaged, please ask the dealer to replace it.

CE-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC
2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

RECYCLING



Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose it in accordance with local laws for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!


Thank you very much for your purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation. The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.



Weelko
the wellness
company

Barcelona
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 938 249 020
E. info@weelko.com
W. www.weelko.com

TECHNICAL SHEET

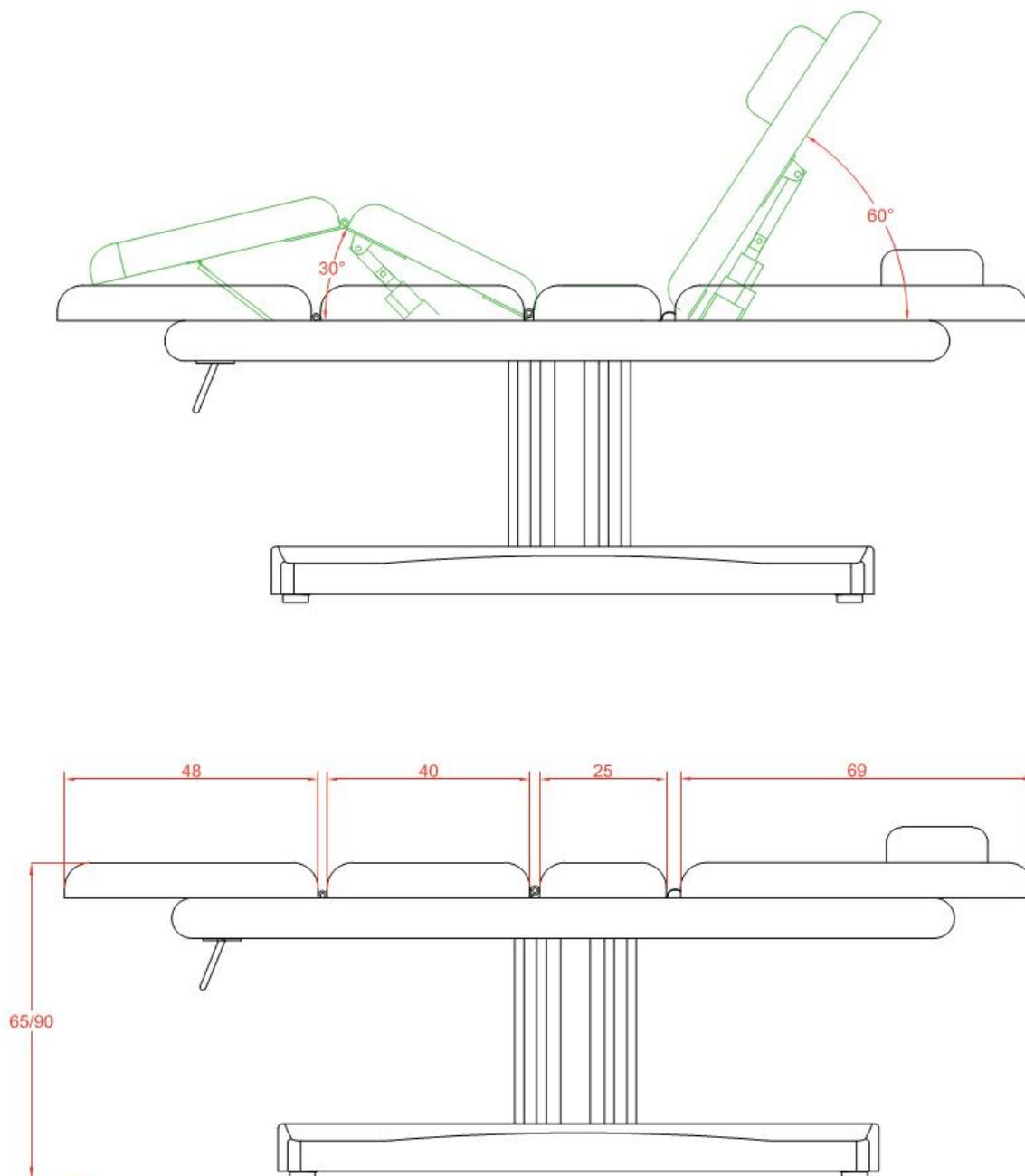

STANDARD FEATURES

Maximum height: 90 cm

Minimum height: 65 cm

Width: 70 cm

Length: 189 cm

Gross weight: 95 kg

Net weight: 83 kg

Packing size: 200 x 82 x 68 cm

Backrest inclination: from 0° to + 60°

Central module inclination or footrest: From 0° to + 30°

Number of motors: 3

Maximum load: 150 Kg

Thickness: 9 cm

Upholstery: High quality polyurethane

Upholstery color: White

Hand controller: Yes, in series

Paper roll holder: Yes, in series

Facial hole: Yes, with cushion

Armrests: No

Arm supports: No

Wheels: No

Heating: No

Electrical characteristics of the motors:

Voltage: 220V~240V

Frequency: 50/60 Hz

Options:

Bed Cover: Ref. COVER 2238TB

Foot controller: Ref. FC-003

Heating


Wheels



Weelko
the wellness
company

Barcelona
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 938 249 020
E. info@weelko.com
W. www.weelko.com

SCHEME SHOWING THE FEATURES OF THE STRUCTURE OF THE BED:

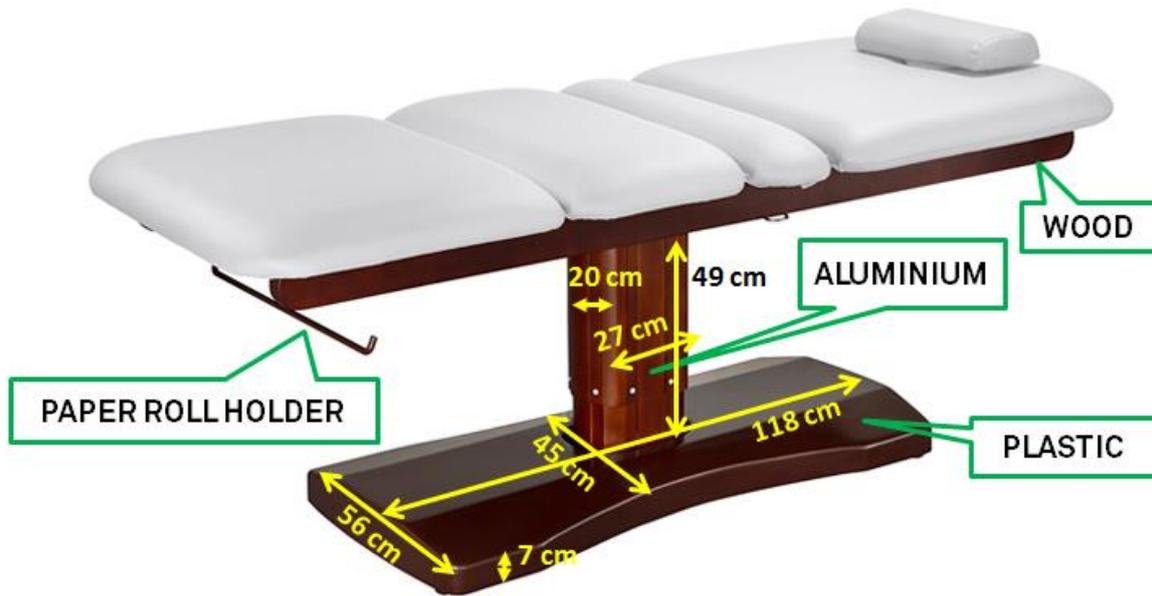

The bed is equipped with a facial hole with a cushion that covers the hole as shown in the picture below.



ASSEMBLY AND INSTALLATION

Your bed is delivered packed with all the components already assembled, ready for use. Open the package and carefully take out the bed between at least two people lifting upwards.

Connect the power cord of the bed to the plug and hang the hand controller on the metallic hook that can be found on the structure where the fixed central module is.

OPERATION

The position of the height, the backrest and the middle section is set using the hand controller (or the foot controller).

Height adjustment. Press the upwards arrow to lift the bed and the downwards arrow to lower it as shown in the picture below.



Backrest adjustment. Press the upwards arrow to lift the backrest and the downwards arrow to lower it as shown in the picture below.



Central section. Press the upwards arrow to lift the central section and the downwards arrow to lower it as shown in the picture below.



PRECAUTIONS

- Be careful not to damage the bed when you take it out from the box.
- Once the bed is placed in the work space, make sure that the power cord is well situated to not hindering the bed performances and the professional's work.
- The frame is designed to take full advantage of the bed for aesthetic and physiotherapy purpose. The bed must not be used for other purposes.
- Do not sit on the edge of the bed.

CLEANING

- Before cleaning the bed, unplug the power cord.
- The frame should be cleaned with a dry cloth or with a slightly damp cloth for the toughest stains.
- Clean the upholstery with a clean and white cloth and water with a gentle soap. Avoid abrasive detergents, use only neutral products.